


**F&P** Brevida™



# NASAL PILLOWS MASK

**Designed for Confidence.  
Built for Comfort.**

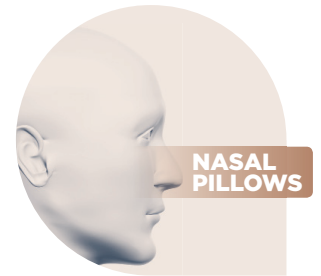

Gaining patient confidence is key to successful CPAP therapy and patient confidence begins with a mask that fits and is comfortable.



# F&P Brevida™ Key features and benefits

## Adjustable Headgear

Adjustable headgear which is designed to stay in place during sleep.

## AirPillow™ Seal

The innovative AirPillow™ inflates to form a gentle and effective seal.




## VisiBlue™

Blue highlights incorporated into the mask key components assist patients with daily mask use.

## Washable Diffuser

The diffuser, which reduces noise and air draft, is washable and durable.

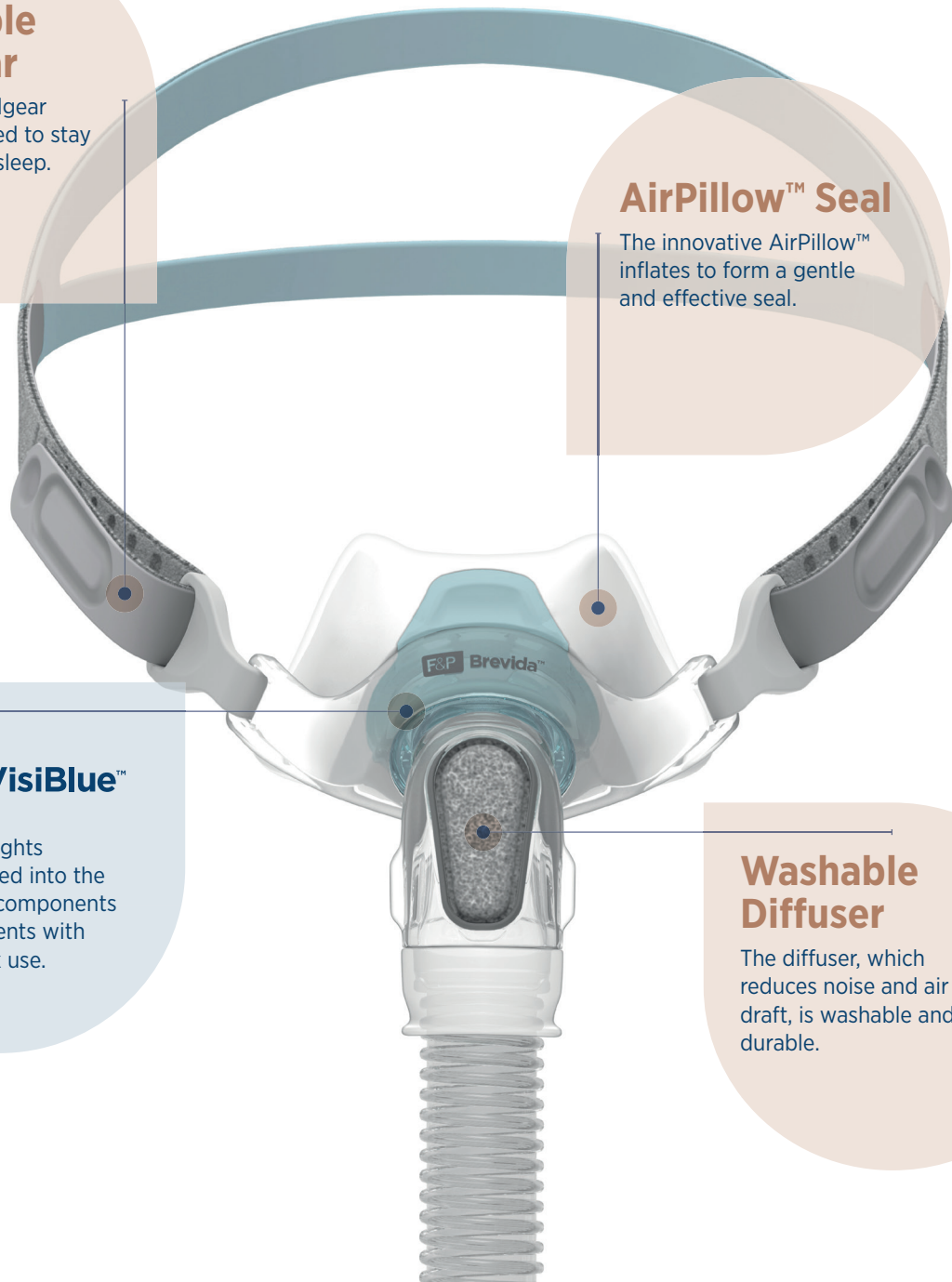

# The innovative F&P Brevida™ AirPillow™ Seal

## Two seal sizes

Extra Small-Small and Medium-Large.

## Extra-thin, soft seal


Sealing region made of extra-thin (0.25 mm) soft silicone to provide a gentle seal.

## Versatile pillows

Versatile pillow geometry to fit a wide range of nostril sizes.

## Stabilizing side supports

Side supports designed to reduce the likelihood of the seal dislodging during sleep.

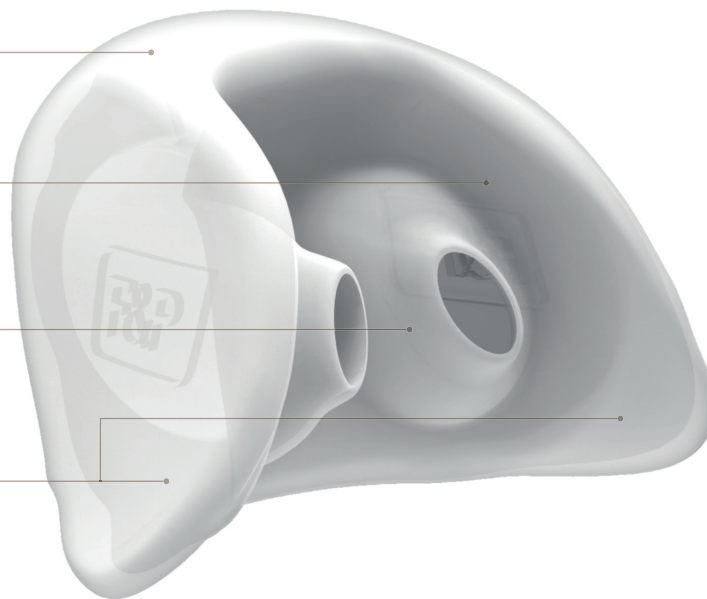


## A gentle, soft seal

The AirPillow™ seal inflates to form a gentle ‘pillow’ of air and seals in and around the nose.


### F&P AirPillow™ Nasal Pillows Seal

Seals in and around  
the nose

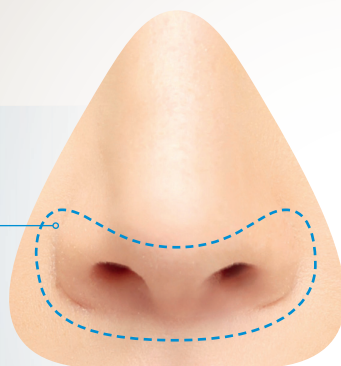

### Traditional Nasal Pillows Seal

Seals directly in the  
nostrils

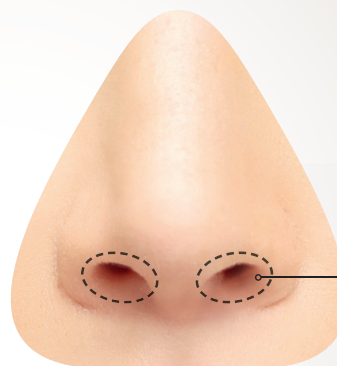

# Fitting your F&P Brevida™



01

**Step 1.**  
**Hold the silicone seal away from your nose** and use your fingers to carefully spread open the soft side supports of the silicone seal and then guide the two openings into your nostrils.



02

**Step 2.**  
**Hold the mask frame in position with one hand** and pull the headgear over your head with the other hand, positioning the straps above the ears as per the fitting image above.



03

**Step 3.**  
**Gently tighten or loosen the headgear straps** on the sides of the headgear until you achieve a comfortable seal.

## NORTH AMERICA/ASIA-PACIFIC REORDER CODES

### MASKS

BREISA Brevida Extra Small-Small (one XS-S seal)  
 BREIMA Brevida Medium-Large (one M-L seal)  
 BREISMA Brevida XS-S and M-L (two seals)

400BRE114 Brevida Seal Medium-Large (50 pack)

400BRE120 Brevida Headgear  
 400BRE151 Brevida Headgear Clips (10 pack)

400BRE161 Brevida Diffuser (10 pack)  
 400BRE171 Brevida Elbow

### SPARE PARTS

400BRE111 Brevida Seal Extra Small-Small  
 400BRE112 Brevida Seal Medium-Large  
 400BRE113 Brevida Seal Extra Small-Small (50 pack)

400FPH241 F&P Swivel 2  
 400BRE131 Brevida Mask No H/G XS-S  
 400BRE132 Brevida Mask No H/G M-L

## EUROPE / AFRICA / SOUTH AMERICA REORDER CODES

### MASKS

BREISU Brevida Extra Small-Small (one XS-S seal)  
 BREIMU Brevida Medium-Large (one M-L seal)  
 BREISMU Brevida XS-S and M-L (two seals)

400BRE114 Brevida Seal Medium-Large (50 pack)

400BRE120 Brevida Headgear  
 400BRE151 Brevida Headgear Clips (10 pack)

400BRE161 Brevida Diffuser (10 pack)  
 400BRE171 Brevida Elbow

### SPARE PARTS

400BRE111 Brevida Seal Extra Small-Small  
 400BRE112 Brevida Seal Medium-Large  
 400BRE113 Brevida Seal Extra Small-Small (50 pack)

400FPH241 F&P Swivel 2